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Self FLOQSwabs® is a self-sampler, that allows you to collect a vaginal sample yourself and send it to laboratory for HPV (human papillomavirus) analysis.

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IMPORTANT NOTES!

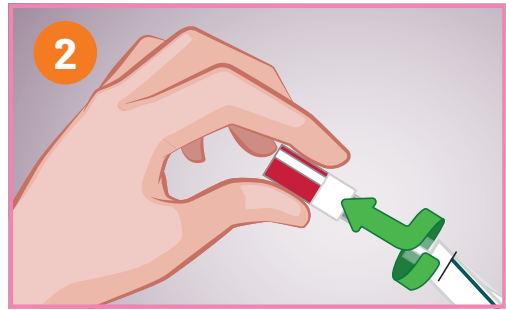
- ✓ Self-sampler should be handled with care and only according to these instructions.
- ✓ Do not use it if the packaging is damaged or if the expiry date has passed.
- ✓ Do not use during menstruation.
- ✓ Do not use during pregnancy or during three months following pregnancy.
- ✓ It is advisable to refrain from intercourse for 24 hours prior to sampling.
- ✓ Do not use any local vaginal treatment for at least two days before using the self-sampler.
- ✓ Vaginal contraceptives, condoms and water-based lubricants can be used as normal.
- ✓ For single use only. Re-use can result in infection and/or incorrect diagnosis.

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
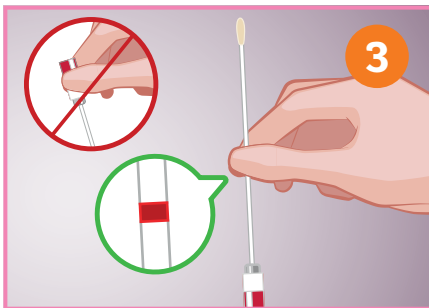
Once you are in a comfortable posi-tion, remove the red cap and pull the swab out of the container. Be careful not to touch any surfaces with the swab.

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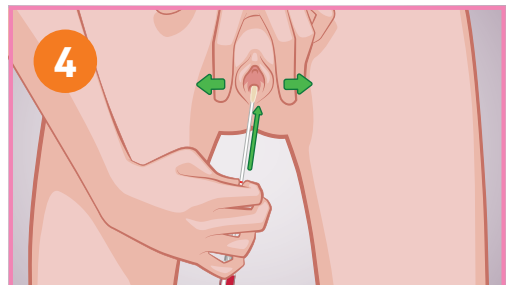
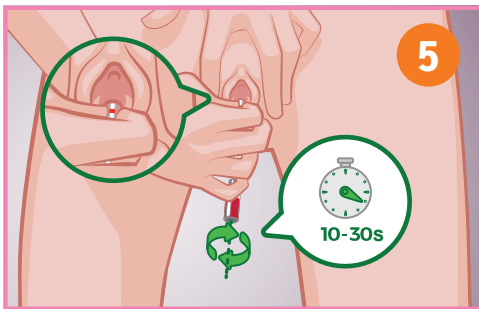
Insert the swab gently into your vagina until you reach the red mark. You do not need to put it in any further than this.

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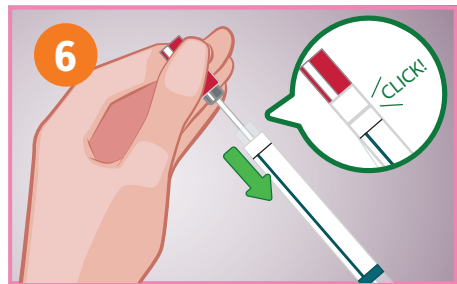Rotate the swab for 10-30 seconds. This should not hurt.

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Gently remove the swab and place it back into the plastic tube. Press the lid to close securely.

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Place the tube into the plastic bag provided and seal it. Place the sample in the prepaid return envelope and mail it preferably **within five days of sampling** (keep the container on room temperature but not in the direct sunlight).